


The iM3 Rodent Soft Tissue Protector and Diamond Disc Safety shield

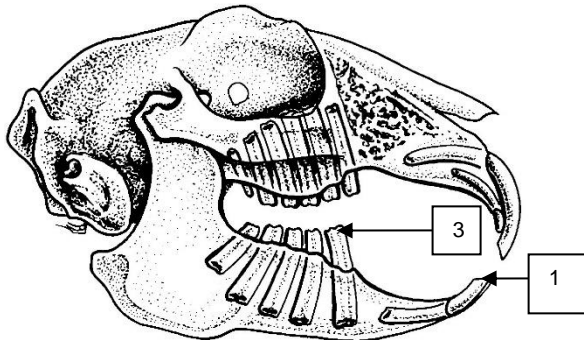

(1) The Diamond Disc Safety Shield (DDSS) is designed to reduce the amount of disc exposed, as a safety precaution. (Stainless Steel)

(2) The Diamond Disc (DD) is used to reduce the length of the tooth and will also produce a smooth surface on fractured teeth. **NOTE: Never run the Disc in reverse (anti-clockwise) as the disc will un-screw from the mandrel and come off.**

(3) The Soft Tissue Protector (STP) allows the Diamond Bur to be used on the back Teeth of Rabbits/Rodents and reduces the risk of soft tissue Lacerations, by exposing only the bottom cutting surface of the Bur. (Stainless Steel)

(4) The 60mm Diamond Bur (DB) is designed to be used to reduce the height of the back teeth of Rabbits and Rodents.

Caution – Exercise great care when using Diamond Discs and Diamond Burs. Always use the appropriate safety equipment.

	<u>ITEM #</u>	<u>CODE</u>	<u>DESCRIPTION</u>
(1)	D2220	Diamond Disc safety shield (DDSS)	
(2)	D2200	Diamond Disc double sided (DD)	
(3)	D2230	Soft tissue protector (STP)	
(4)	D2235	Diamond Bur X - Long (DB)	

The complete Kit includes 1 x Double sided Diamond disc, 1 x Long Diamond bur, 1 x Soft tissue protector and Diamond disc protector.

To be used with air driven dental equipment:



(A) L6200 Advantage Low Speed Air Driven Motor (shown) (B) L6860 1:1 Straight Nose Cone