

iM3 EDTA

File Lubricant Cream
For veterinary use only



iM3 EDTA Cream is a dental cleaner for the removal of pulp tissue in the root canal. It removes inorganic matrix through good chelation and makes it easier to use files or reamers thanks to its lubrication effect.

Application Range

- Root canal preparation with files and reamers.
- Presence of calcification in the canal.
- Smear layer removal.

Application method

- Place disposable tip on iM3 EDTA syringe.
- Deposit a proper amount in the canal orifice.
- Carefully pump into the canal with a fine file or reamer.
- Rinse off iM3 EDTA by thorough irrigation with sodium hypochlorite.

Package

- 2x 6g Syringe of iM3 EDTA File Lubricant Cream
- 5x disposable applicator tips

Cautions

- iM3 EDTA is only used for veterinary dental applications.
- Avoid contact with eyes.
- In case of contact with the skin or eyes, clean with pure water several times, and then consult with a medical specialist if necessary.
- Do not use expired iM3 EDTA.
- For Veterinary use only.

Storage

- Do not store under intense light or high temperature.
- Close the cap immediately after use.
- Store in a dry and cool place.
- Valid for 2 years from the manufacturing date.

Component

- EDTA: 15wt%
- Eylene glycol >65wt%



THE VETERINARY DENTAL COMPANY

USA (800) 664 6348 info@im3usa.com

Australia +61 2 9420 5766 sales@im3vet.com

Ireland +353 (0)1 6911277 sales@im3dental.com