

# iM3 Soft Impression Putty

*Two-part impression material*

*For veterinary use only*



iM3 Soft Impression Putty is hydrophilic addition polymerization type silicon rubber impression material with mixing base & catalyst.

## **Application Range:**

- For impression taking
- For individual tray
- Impression taking of crown and bridge
- Impression taking of weak and old teeth

## **Application Method:**

- Take out same amount of Base and of Catalyst (1:1 volume)
- Knead Base and Catalyst together by hand until a mixed colour is attained. Wear disposable vinyl gloves.
- Apply the mixed material to the tray and set into the mouth.
- After the material is perfectly set, remove from the mouth

Mixing time - 23°C - 40 seconds

Working Time - 23°C - 20 minutes

Setting time - 32°C - 4 minutes

Hardness (Shore A) - 70

## **Storage:**

- Close the cap immediately after use
- Store between 18- 24°C



THE VETERINARY DENTAL COMPANY

USA (800) 664 6348 info@im3usa.com

Australia +61 2 9420 5766 sales@im3vet.com

Ireland +353 (0)1 6911277 sales@im3dental.com