


## Proper use of the **iM3**<sup>®</sup> iPush

Order Code  
L7050

Push the lever just enough  
to depress the button, do  
not use excessive force

When pressed correctly  
the bur will glide in and out  
of the handpiece with no  
resistance

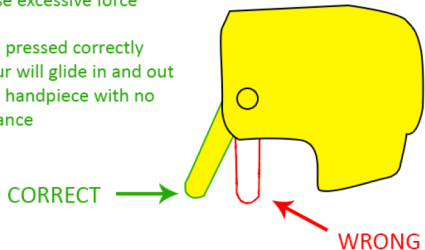


If you have pushed the lever down this much  
it's too hard & may break!

# **iM3**<sup>®</sup>

www.im3vet.com  
EC Patent No. 001882788  
USA Patent No. 29/398937

## Proper use of the **iM3**<sup>®</sup> iPush

Order Code  
L7050

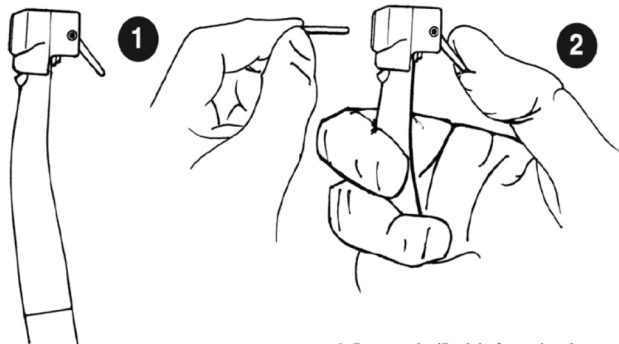

1. Place iPush onto the LED Advantage Push button handpiece.
2. Depress the lever with your thumb and insert the bur all the way until it stops.

3. Remove the iPush before using the Handpiece.
4. To change the Bur repeat step 2. and remove the Bur.  
To increase the life of the Handpiece, Oil between each use and replace the Bur between each patient. See manual

# **iM3**<sup>®</sup>

www.im3vet.com  
EC Patent No. 001882788  
USA Patent No. 29/398937