




Are your 42-12 Scaler Tips due to be replaced?

Save Time and Money with efficient scaling!

- All Scaler Tips wear out!
- Worn tips result in reduced vibration on the tooth!
- Worn tips result in inefficient scaling and frustration!
- Worn tips result in poor displacement of water to aid cooling!
- Worn tips result in excess anesthesia and procedure time and money!
- During the working life of the Scaler Tip, display lights and Water Flow will require adjusting!
- Keep the Ferrite Rod from a Worn Tip as a valuable spare!

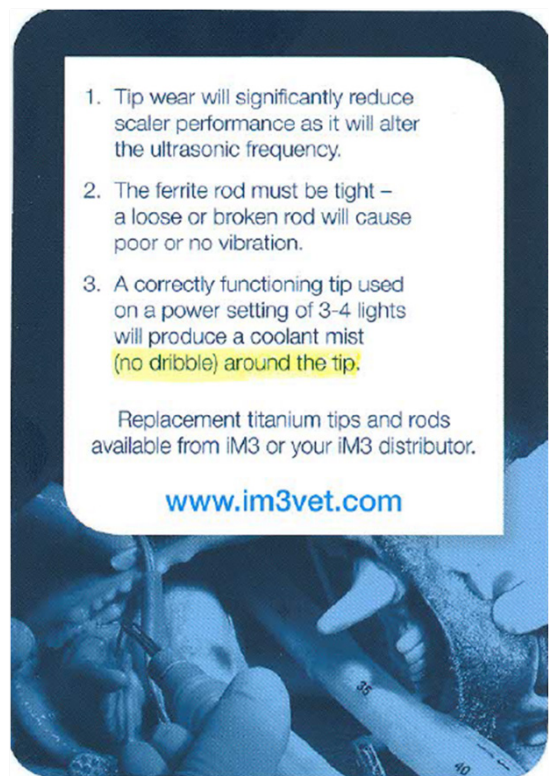

1. Tip wear will significantly reduce scaler performance as it will alter the ultrasonic frequency.
 2. The ferrite rod must be tight – a loose or broken rod will cause poor or no vibration.
 3. A correctly functioning tip used on a power setting of 3-4 lights will produce a coolant mist (no dribble) around the tip.
- Replacement titanium tips and rods available from IM3 or your IM3 distributor.

www.im3vet.com

Ferrite Rod

O6860

For the iM3 42-12 Scaler Tip Insert



42-12 Scaler Tip Insert

O6840
Universal Tip

O6850
Perio Tip

O6845
Thinline straight tip is ideal for felines and small mouths.



VET DENTAL UNITS | VET DENTAL X-RAY | ULTRASONIC SCALERS | HAND INSTRUMENTS | ACCESSORIES & CONSUMABLES



THE VETERINARY DENTAL COMPANY

iM3 Dental Limited

Main Office +353 (0) 1 6911277 | UK Direct Number +44 (0) 1423 224297

e sales@im3dental.com | www.im3vet.eu

How to Change a Broken Ferrite Rod

