

# iM3 - TTMG T2000


## Table Top Mouth Gag (Rabbit/Rodent)

The TTMG was designed by iM3 with the help of specialists to hold the anaesthetised rabbit or rodent in the best possible position for examination and treatment of the oral cavity without assistance.


- Adjustable height S/S gags to suit any size rabbit or rodent
- Adjustable table height ensures the best viewing angle of the oral cavity
- The S/S gag & silicon mat are easy to clean and autoclavable 121 Deg C. Remove black plastic feet from S/S legs before autoclaving. **NOTE Blue plastic base is not Autoclavable.**
- Quality components and innovative design for ease of use.

### Optional

- LED Light for improved illumination of the oral cavity.

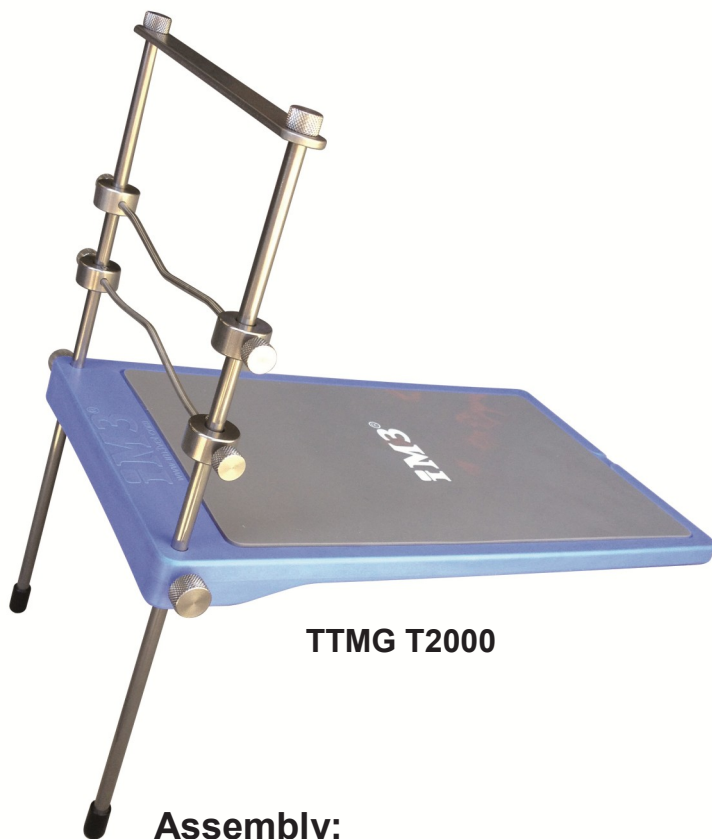

TTMG T2000

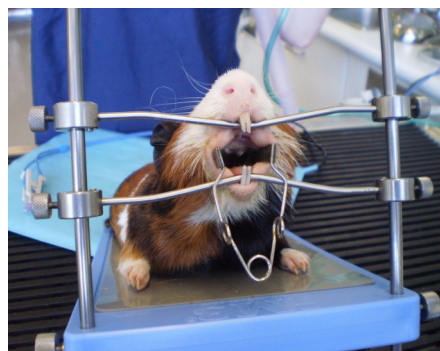

### Assembly:

Slide the two S/S legs of the complete gag assembly through the two top holes in the top of the blue plastic platform. Loosen the knurled knobs on either side to allow the legs to slide through and tighten at the desired height. Push the black plastic feet onto the end of the S/S legs. Remove these feet for autoclaving. Adjust the height of the mouth gags by loosening the knurled knobs on the side of the gags. Tighten knob in desired position.

VET DENTAL UNITS | VET DENTAL X-RAY | ULTRASONIC SCALERS | HAND INSTRUMENTS | ACCESSORIES & CONSUMABLES



THE VETERINARY DENTAL COMPANY

iM3 Dental Limited

Main Office +353 (0) 1 6911277 | UK Direct Number +44 (0) 1423 224297

e sales@im3dental.com | [www.im3vet.eu](http://www.im3vet.eu)