iM3® iM3 Ultra S | ST

INSTRUCTIONS FOR USE AND
TECHNICAL MANUAL

Contents

Symbols	4
1. Introduction	5
2. Electromagnetic compatibility (EMC)	7
3. Unpacking	8
4. Scope of delivery	9
5. Safety notes	10
6. Description	13
Control unit U8200.....	13
Control unit U8300.....	14
LED Handpiece	15
Foot Control U8007	16
7. Start-up	17
8. Operation	19
Handpiece Assembly/Removal.....	19
Changing the tip	20
Control unit - Startup	23
Control unit - Functions	24

Contents

9. Error messages	25
10. Hygiene and maintenance	26
11. Maintenance	27
Manual cleaning	28
Inspection, Maintenance and Testing	30
Exchanging the supply hose O-rings	31
U8300	32
12. Servicing	33
13. iM3 accessories and spare parts	35
14. Technical data	36
15. Disposal	38
Explanation of warranty terms	39
Authorised iM3 service partners	40

Symbols

	Consult Instructions for Use		CE mark	V	Supply voltage
	Follow Instructions for Use		Foot control	W	Electric power consumption
	Date of manufacture		Non-ionizing electromagnetic radiation	Hz	Frequency of the alternating current
	Do not dispose of with domestic waste		Catalogue number		ESI (External System Interface)
	DataMatrix Code for product information including UDI (Unique Device Identification)		Serial number		Coolant volume
	WARNING! (risk of injury)		DC – direct current		Upper limit of temperature <small>+30°C Max. (+86°F)</small>
	ATTENTION! (to prevent damage occurring)		General explanations, without risk to persons or objects		Sterilisable up to the stated temperature
			Type B applied part (not suitable for intracardiac application)		Thermo washer disinfectable

1. Introduction

For your safety and the safety of your patients

These Instructions for Use explain how to use your device. However, we must also warn against possible hazardous situations. Your safety, the safety of your team and, of course, the safety of your patients are of paramount importance to us.

Observe the safety notes.

Intended use

U8200, U8300:

Drive unit with a piezoceramic oscillating system, which moves the tip in a linear oscillation. The drive unit is used for the removal of supragingival calculus and subgingival concretions and for endodontic application and preparation of tooth enamel.

Foot control for operation of the electrical equipment.

FOR VETERINARY DENTAL USE ONLY.

Misuse may damage the device and hence cause risks and hazards for user and third parties.

Introduction

Responsibility of the manufacturer

The manufacturer can only accept responsibility for the safety, reliability and performance of the device when it is used in compliance with the following directions:

- > The device must be used in accordance with these Instructions for Use.
- > Only the components approved by the manufacturer may be replaced (O-ring, coolant filter, pump cartridge).
- > Modifications or repairs must only be undertaken by an authorised iM3 service partner (see page 40).
- > The device has no components that can be repaired by the user.
- > The electrical installation at the premises must comply with the regulations laid out in IEC 60364-7-710 (>Installation of electrical equipment in rooms used for approved purposes«) or with the regulations applicable in your country.
- > Unauthorised opening of the device invalidates all claims under warranty and any other claims.

Improper use, unauthorised assembly, modification or repair to the device, non-compliance with our instructions or the use of accessories and spare parts which are not approved by iM3, invalidates all claims under warranty and any other claims.

FOR VETERINARY DENTAL USE ONLY.

2. Electromagnetic compatibility (EMC)

Electrical equipment is subject to particular precautions in regard to EMC and must be installed and put into operation in accordance with the EMC notes included.

iM3 guarantees the compliance of the device with the EMC requirements only when used with original iM3 accessories and spare parts. The use of accessories and spare parts not approved by iM3 can lead to an increased emission of electromagnetic interference or to a reduced resistance against electromagnetic interference.

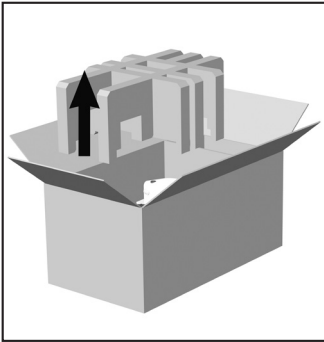
HF communication equipment

Portable HF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (11.8 inches) to the device. Otherwise, degradation of the performance of this device could result.


The device may be interfered by other equipment, even if these other devices comply with CISPR (International special committee on radio interference) emission requirements.

The device is not intended for use in the vicinity of HF surgical devices.


3. Unpacking

❶ Remove the insert.

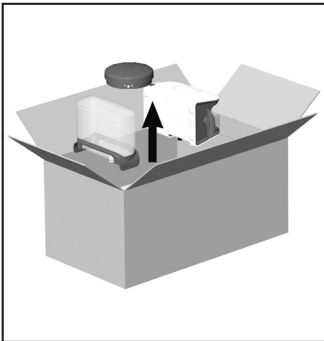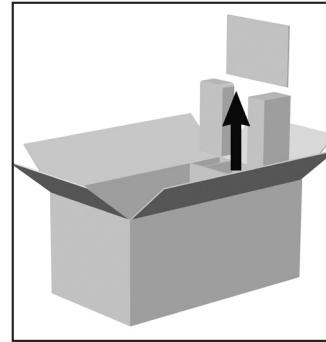❷ Remove the control unit, the coolant tank (U8300) and the foot control.

❸ Remove the Instructions for Use and the accessories.

iM3 packaging is environmentally friendly and can be disposed of by industrial recycling companies. However, we recommend that you keep the original packaging.

4. Scope of delivery

	Control unit (100–230 V)	U8200 ULTRA-S	U8300 ULTRA-ST
REF 02675000	Coolant filter	X	
REF 05075600	Coolant hose (Ø 6 mm, approx. 2 m)	X	
REF 08016690	Power supply with adaptor	X	X
REF 07991190	Coolant tank		X

- > Only use tips that have been approved by iM3 and the associated tip changers.
- > An overview of the correct power settings is included with every tip.
- > With periodontal tips, the device is suitable for the removal of concretions in the subgingival region, but not for applications which demand sterile conditions. Choose the lower performance range when carrying out periodontal treatments on hypersensitive patients in order to guarantee optimum pain-free treatment.
- > Ensure that the original shape of the tips is not affected (e.g. by being dropped).
- > The tips must not be bent back into shape or reshaped.
- > Locate and secure the tip only with the device switched off.
- > Never touch the tips when vibrating.
- > Insert the tip changer onto the inserted tip of the stationary device after every treatment (protection against injury and infection, tip protection). Tips that are changed using a spanner must be removed from the device immediately after treatment.
- > Do not touch into the tip changer (with tip inserted).
- > Check for the effect of wear on the tips using the accompanying tip card.
- > Change tips if there are visible signs of wear.

Control unit U8300

- > Never fill or top up the coolant tank with liquids hotter than 30°C/86°F.
- > Replace a faulty or leaky pump cartridge immediately.

Control unit U8200, U8300

Risks due to electromagnetic fields

The functionality of implantable systems, such as cardiac pacemakers and implantable cardioverter defibrillators (ICD) can be affected by electric, magnetic and electromagnetic fields. This device complies with the reference values defined in EN 50527-2-1/2016 for unipolar and bipolar pacemakers and is therefore suitable for patients with pacemakers.

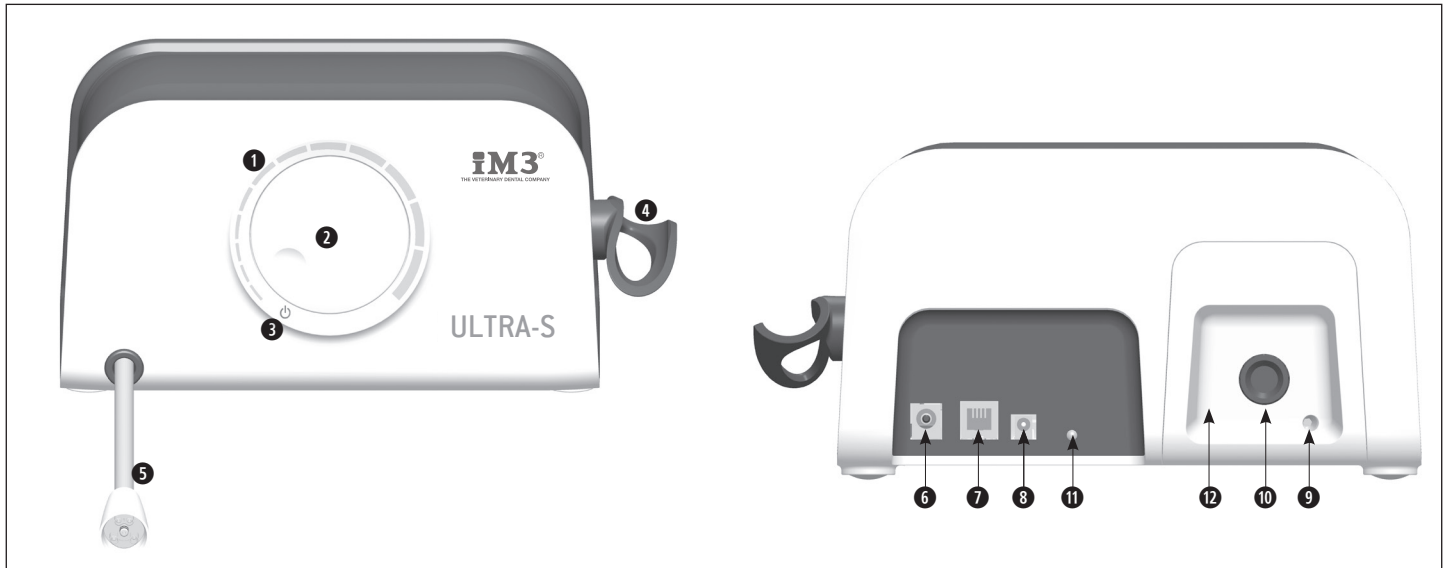
- > Find out if patient and user have implanted systems before using the device and consider the application.
- > Keep a safe distance of at least 10 cm (3.94 inch) between the device and the cardiac pacemakers.
- > Make appropriate emergency precautions and take immediate action on any signs of ill-health.
- > Symptoms such as raised heartbeat, irregular pulse and dizziness can be signs of a problem with a cardiac pacemaker or ICD.

The control unit is designed for use with the iM3 handpiece U8205 so only this is to be used with the control unit. The use of other handpieces could lead to a malfunction or destruction of the electronics.

6. Description


Control unit U8200

1	Power range	Connections	11	Status LED
2	Power regulator		6	Power supply
3	>>OFF<<		7	ESI (external service interface)
4	Handpiece support (adjustable)		8	Foot control
5	Supply hose		9	Coolant hose
		10	Coolant regulator	
			12	Cover

Description

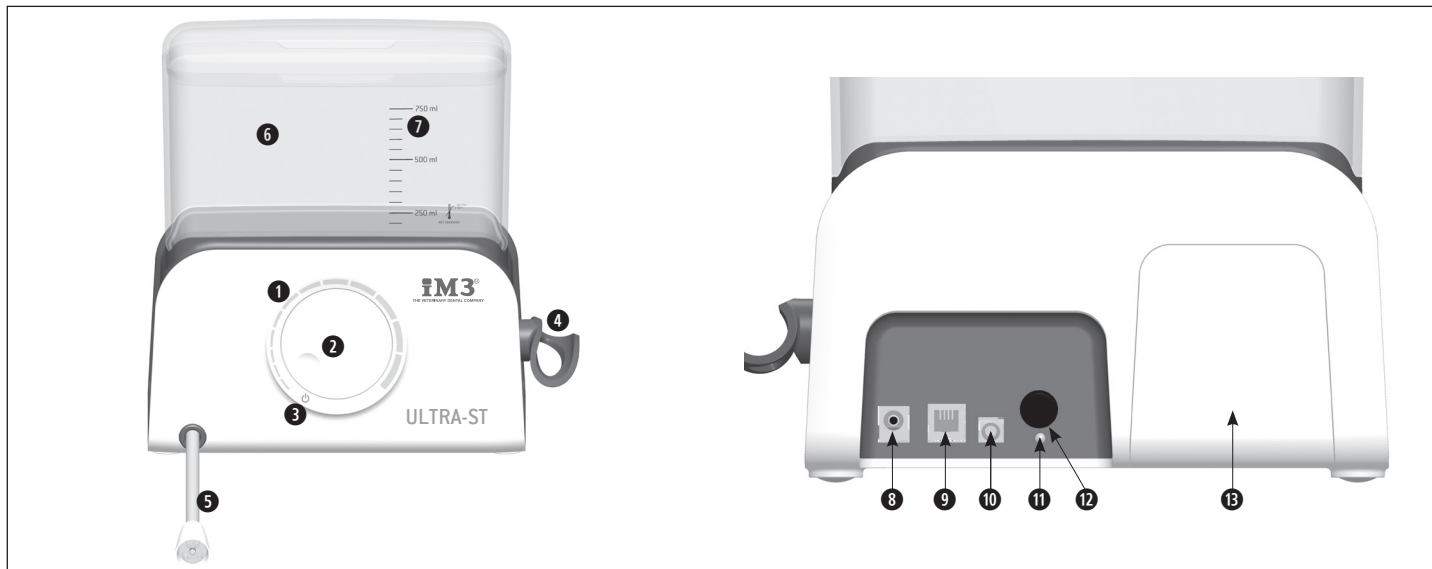
Control unit U8300

1	Power range	6	Coolant tank	11	Status LED
2	Power regulator	7	Filling level indicator	12	Coolant regulator
3	>>OFF<<	Connections		13	Cover
4	Handpiece support (adjustable)	8	Power supply		
5	Supply hose	9	ESI (external service interface)		
		10	Foot control		

Description

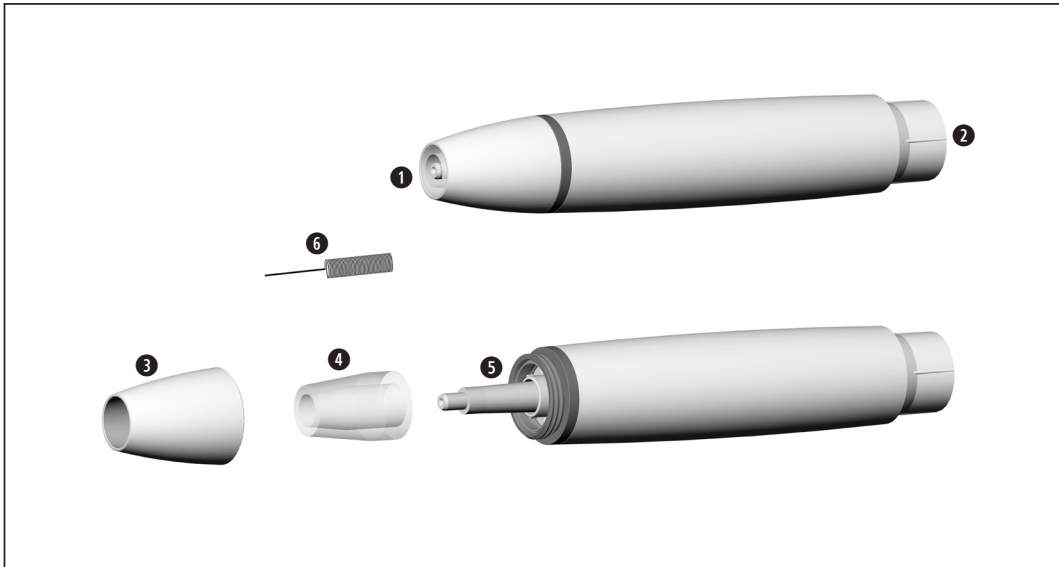LED Handpiece

1	Thread	4	Optical fibre
2	Connection for supply hose	5	Optic outlet
3	Handpiece cap	6	Nozzle Cleaner

7. Start-up

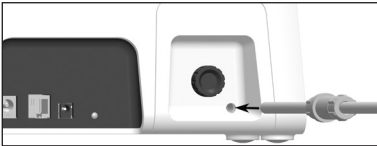
Control unit general

Ensure that the device can be disconnected from the power supply at any time.


Place the device on a flat, level surface.

Control unit U8200

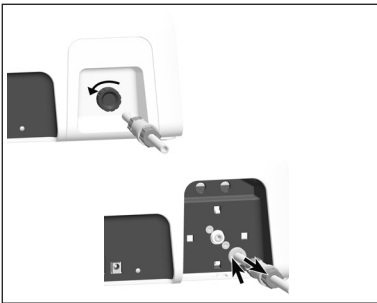- 1 Push the coolant hose until the limit stop.

Control unit U8200

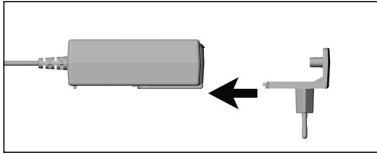
Remove the coolant hose

- 1 Screw off the coolant regulator.
- 2 Uncrew the cover and remove it.
- 3 Push the connection ring and simultaneously remove the coolant hose.


Control unit U8300 Coolant tank

- 1 Fill the coolant tank and attach it. The coolant tank snaps audibly into place.


Control unit

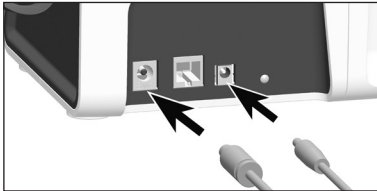- 1 Slide the adapter onto the power supply.

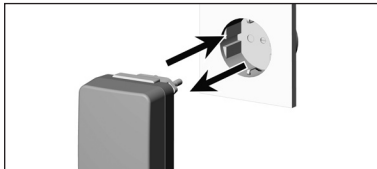
- 2 Connect the power supply.
- 3 Connect the foot control.

- 4 Plug the power supply into a socket.
- 5 Pull the power supply out of the socket.

8. Operation

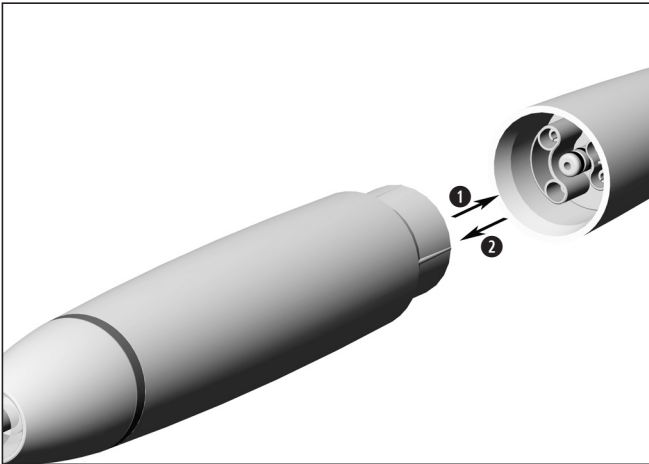
Assembly/Removal

❶ Push the handpiece onto the supply hose.

👉 Note the positioning.

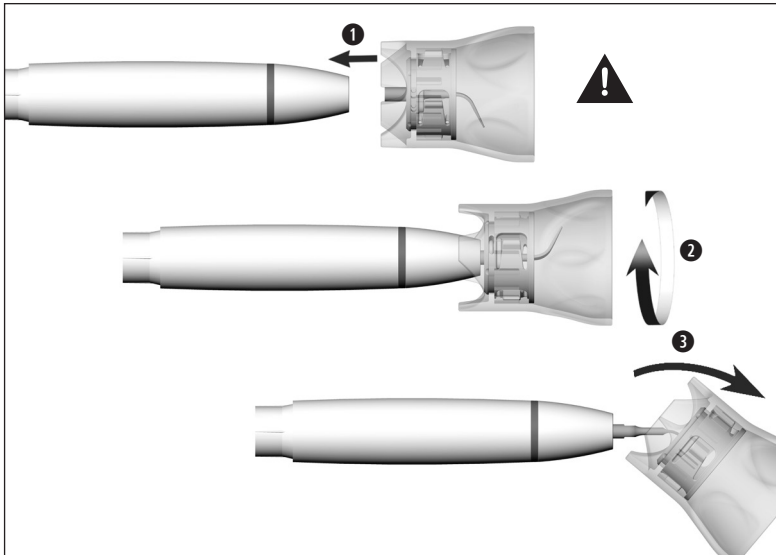❷ Remove the handpiece.

Insert tip with tip changer

Ensure the matching thread system
(at the handpiece, tip changer, tip)!

- 1 Position the tip on the thread of the handpiece.
- 2 Turn the tip changer until it audibly engages.
- 3 Withdraw the tip changer.

Verify full engagement.

Press the tip with about 1 N (= 100 g) pressure onto a firm object to test the loading capacity of the tip.

Do not hold the handpiece at eye level!

- > Attach the handpiece to the supply hose.
- > Insert the tip.
- > Put the device into operation.

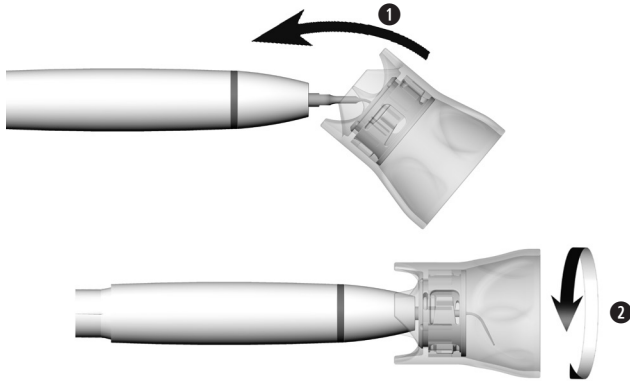
In the event of operating malfunctions (e.g., vibrations, unusual noise, overheating, coolant failure or leakage) **stop the device immediately** and contact an authorised iM3 service partner.

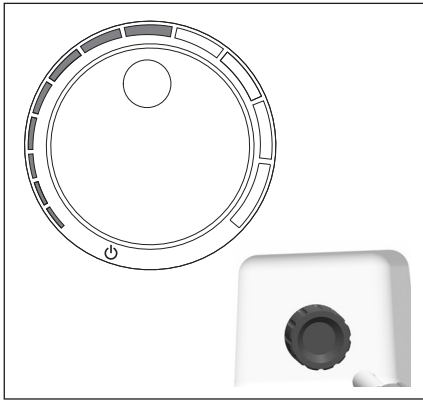
Remove tip with tip changer

- 1 Place the tip changer onto the tip.
- 2 Unscrew the tip with the tip changer.

Leave the tip in the tip changer until the hygienic maintenance process!


Control unit

- ❶ Attach the handpiece to the supply hose. Insert tip.
☞ Follow the directions and safety notes in the Instructions for Use of the iM3 handpieces.
 - ❷ Set power and coolant (variable)
 - ❸ Press foot control
- > Release foot control: Fade-out time of the handpiece LED 30 seconds

 > Before every patient: Perform rinsing function for automatic internal cleaning of the coolant channels.

Ultra	U8200	U8300
Rinsing function for automatic internal cleaning of the coolant channels	✓	✓

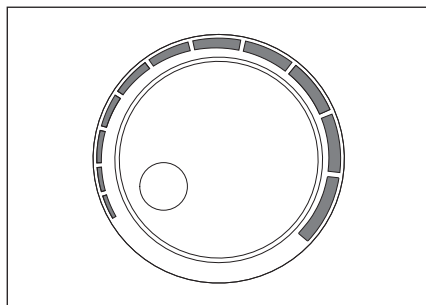Control Unit - Rinsing function

Control unit U8200, U8300

- 1 Remove the handpiece from the supply
 - 2 Set power between off and 0
 - 3 Press foot control 3 times within 3 seconds
- > Rinsing function active for 30 seconds

 Cancelling the rinsing function with one of these options:

- > Press foot control
- > Adjust power regulator

9. Error messages

Control Unit U8200, U8300

The error messages are indicated at the rear of the control unit by the status LED (flashing green).

Flashing cycle	Description of error	Solution
1x	Overheating	<ul style="list-style-type: none">> Switch off control unit> Allow the control unit to cool for at least 10 minutes> Observe permissible ambient temperature/operating mode
2x	Foot control	<ul style="list-style-type: none">> Release foot control
5x	Time-out (> 15 min)	<ul style="list-style-type: none">> Release foot control (must not be active for longer than 15 minutes without interruption)
6x	Handpiece	<ul style="list-style-type: none">> Check tip (full engagement, damage, torque)> Dry the handpiece/supply hose> Check plug-in connection of the handpiece/supply hose> If the error message appears again, contact an authorised iM3 service partner immediately.
8x	System error	<ul style="list-style-type: none">> Start the device again> Contact an authorised iM3 service partner.

 Follow your local and national laws, directives, standards and guidelines for cleaning, disinfection and sterilisation.

> Wear protective clothing, safety glasses, face mask and gloves.

> Wipe the entire device and the foot control with disinfectant.

> Ensure that no fluids enter the device.

Recommended sterilisation cycles


> Steam sterilisation (type B, S, N)

> Sterilisation time at least 3 minutes at 134°C (273°F), 4 minutes at 132°C (270°F), 30 minutes at 121°C (250°F)

> Maximum sterilisation temperature 135°C (275°F)


11. Maintenance

Ultra	U8200	U8300
Replacing the O-ring of the coolant tank	–	✓
Replacing the coolant filter of the coolant hose	✓	–
Replacing the pump cartridge	–	✓

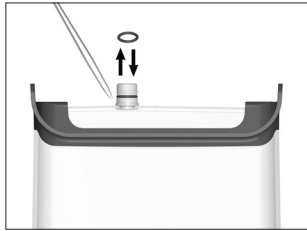Replacing the O-ring of the coolant tank

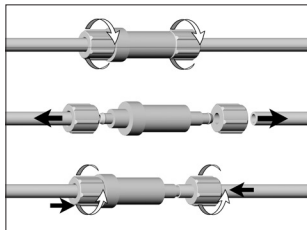- 1 Remove the O-ring with tweezers.
- 2 Slide on the new O-ring.


Replacing the coolant filter of the coolant hose


Replace the coolant filter if it is soiled or after 1 year at the latest.

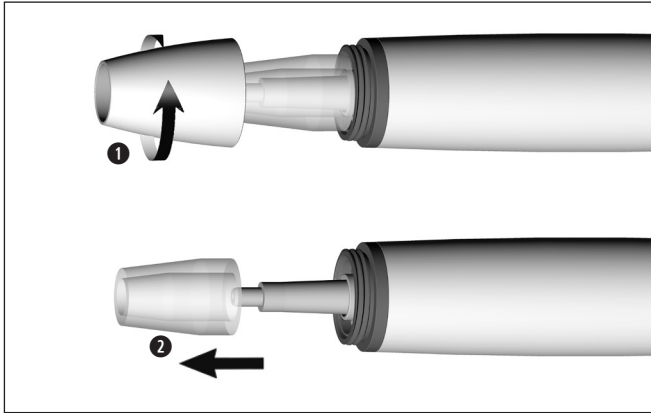- 1 Unscrew the cap nut from the coolant filter.
- 2 Pull off the coolant hose from the coolant filter.
- 3 Attach the coolant hose through the cap nut onto the new coolant filter. Screw the cap nut tight.

Disassembling the device

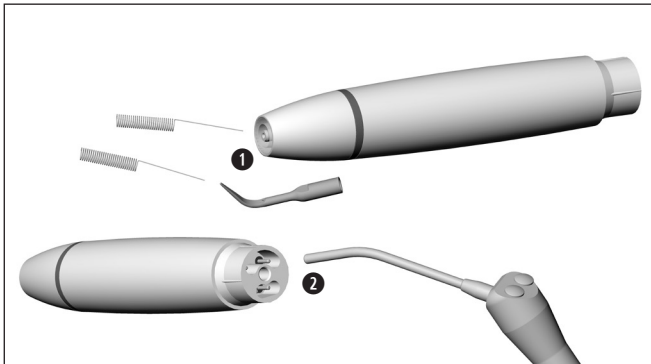- 1 Unscrew the handpiece cap.
- 2 Remove the optical fibre.

Cleaning the spray nozzles

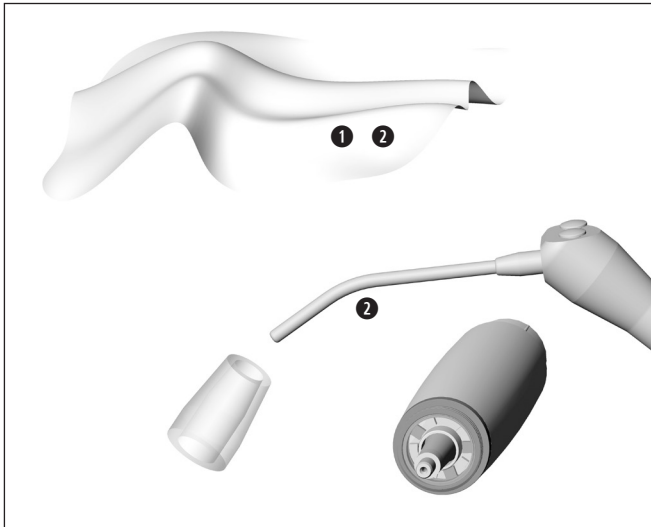
- 1 Clean coolant outlets carefully with the nozzle cleaner to remove dirt and deposits.

 The nozzle cleaner can be cleaned in an ultrasonic bath and/or in the washerdisinfector.

Cleaning the coolant tube

- 2 Blow through the coolant tube using compressed air.

 In the case of clogged up coolant outlets or coolant tubes contact an authorised iM3 service partner.

Cleaning the optic outlet and the optical fibre

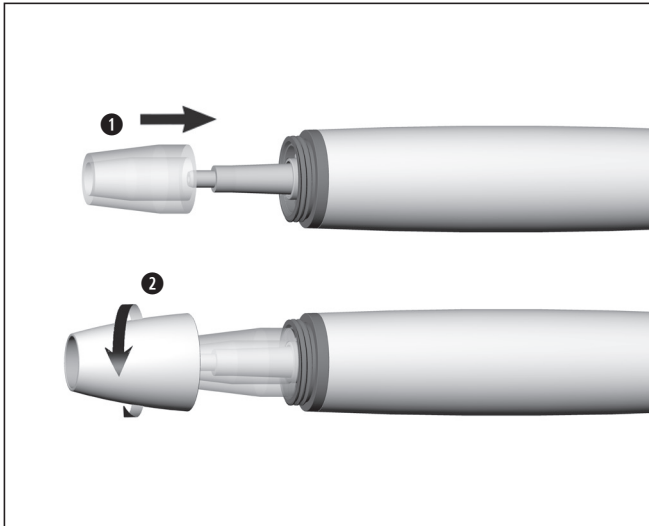
Avoid scratching the optic outlet and the optical fibre!

- 1 Wash the optic outlet and the optical fibre with cleaning fluid and a soft cloth.
- 2 Blow the optic outlet and the optical fibre dry using compressed air or dry it carefully with a soft cloth.

Carry out a visual inspection after each cleaning process.
Do not use the device if the optic outlet or the optical fibre is damaged and contact an authorised iM3 service partner.

Reassembling the device

Reassemble the device following cleaning and disinfection.

①

Fit optic fibre onto device.

②

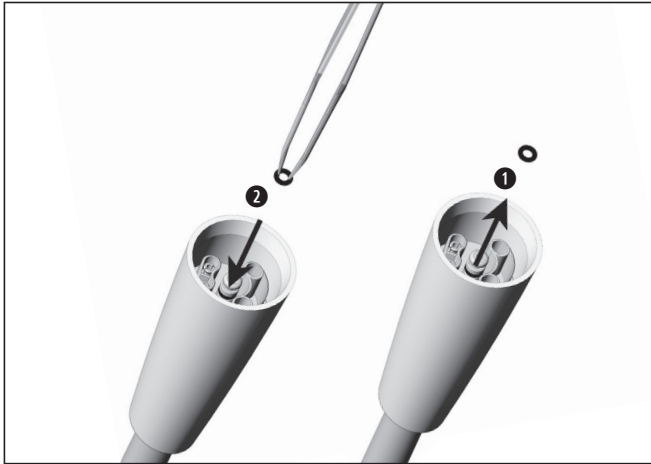
Screw on the handpiece cap.

Sterilise the handpiece, the tip and the tip changer following cleaning and disinfection.

Maintenance

Exchanging the supply hose O-rings


- ❶ Remove O-rings.
- ❷ Slide on the new O-rings with a pair of tweezers.


 Always change all O-rings to ensure tightness.

Maintenance U8300

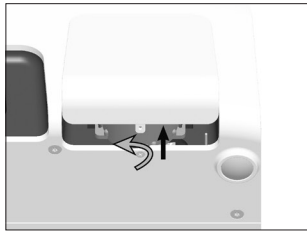
Replacing the pump cartridge


① Uncrew cover and remove.

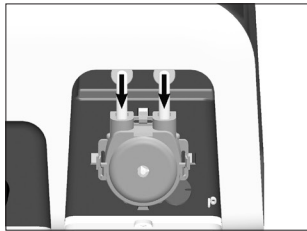② Pull off coolant hoses.

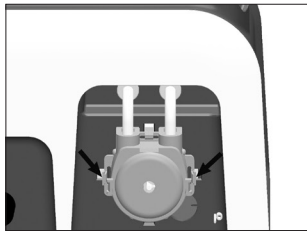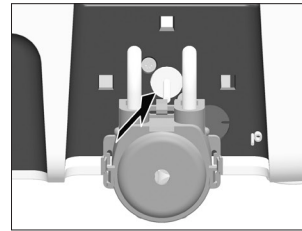
③ Unlock pump cartridge and pull it out.


④ Attach new pump cartridge.

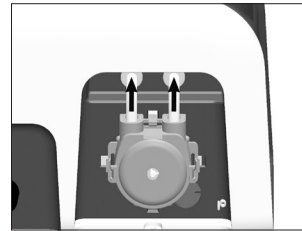 Snaps audibly into place.

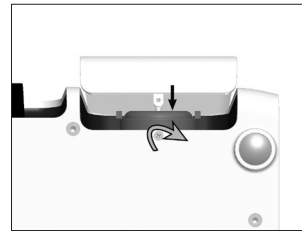⑤ Attach the coolant hoses until the limit stop.

⑤ Attach cover and screw tight.

12. Servicing

Regular checks

Regular servicing of function and safety including the accessories is necessary and should be carried out at least once every three years, unless shorter intervals are prescribed by law.

The inspection must be undertaken by a qualified organisation and must include the following procedures:

Control unit

- > External visual inspection
- > Measurement of the device leakage current
- > Measurement of patient leakage current
- > Visual inspection of internal components on suspicion of safety interference, e.g., mechanical damage of the enclosure or indicators of overheating

Foot control

- > External visual inspection
- > Function test with check to see if the maximum speed can be reached

Processing cycles

- > We recommend replacing the tip changer after 250 processing cycles
- > Check signs of wear on the tips (see tip card)

The regular checking must only be performed by an authorised iM3 service partner.

Servicing

Repair and returns

In the event of operating malfunctions immediately contact an authorised iM3 service partner.
Repairs and maintenance work must only be undertaken by an authorised iM3 service partner.

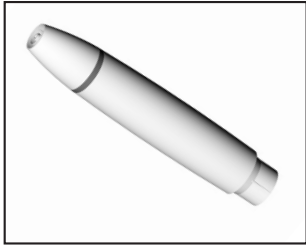
> Always return equipment in the original packaging!


13. iM3 accessories and spare parts

Use only original iM3 accessories and spare parts or accessories approved by iM3.
Suppliers: iM3 partners

U8205
Handpiece

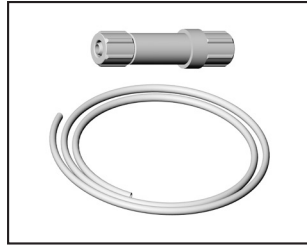
Coolant filter


U8302
Coolant tank

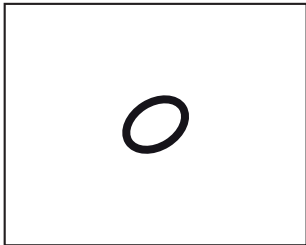
U8007
Foot control

U8303
O-ring for coolant tank

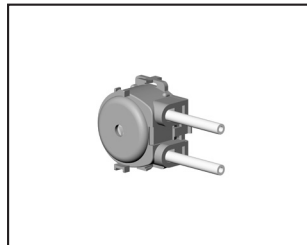U8304
Pump cartridge

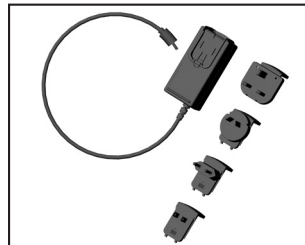U8305
Power supply
with adaptor

14. Technical data

Control unit	U8200	U8300
Power supply:	28.3–31.5 V _{DC}	
Mains voltage:	100–230 V	
Nominal current:	max. 830 mA	
Permissible voltage fluctuation:	± 10%	
Max. output power to the handpiece under load (ultrasonic):	12 W	
Frequency (ultrasonic):	22–35 kHz	
Operating mode:	S3 (14sec/6sec)	
Max. oscillation amplitude (Tip 1U):	0.2 mm	
Max. water pressure:	1–6 bar	
Max. coolant flow (adjustable):	ca. 50 ml/min	
Dimensions in mm (WxDxH):	120 x 185 x 110	120 x 185 x 205
Weight:	807 g	1,064 g

Ambient conditions

Temperature during storage and transport:

-20°C to +60°C [-4°F to +140°F]

Humidity during storage and transport:

8% to 80% [relative], non-condensing

Temperature during operation:

+10°C to +35°C [+50°F to +95°F]

Humidity during operation:

15% to 80% [relative], non-condensing

Technical data

		U8205 LED HANDPIECE
Max. power output to the handpiece with load (ultrasonic)	[W]	10
Frequency (ultrasonic)	[kHz]	22–35
Minimum coolant supply volume	[ml/min]	0*/20
Maximum coolant supply volume	[ml/min]	50
Water pressure	[bar]	1–6
Max. oscillating amplitude (Tip 1U)	[mm]	0.2

* for tips where no coolant is used

U8200 & U8300 is manufactured in conformity with the appropriate standards and complies with the following directives:

IEC 60601-1-2:2007, IEC 61000-4-2, IEC 61000-4-4, IEC 61000-4-5, IEC61000-4-11, IEC 61000-4-8, IEC 61000-4-6, IEC 61000-4-3, IEC 60601-1-2:2014, CISPR 1, IEC 61000-3-2 (*), EC 61000-3-3 (*)

(*) Remark: for devices with power consumption of 75 W to 1000 W only

15. Disposal

Ensure that the parts are not contaminated on disposal.

Follow your local and national laws, directives, standards and guidelines for disposal.

- > Device
- > Waste electrical equipment
- > Packaging

Explanation of warranty terms

This iM3 device has been manufactured with great care by highly qualified specialists. A wide variety of tests and controls guarantees faultless operation. Please note that claims under warranty can only be validated when all the directions in the Instructions for Use have been followed.

As the manufacturer, iM3 is liable for material or manufacturing defects within a warranty period of 24 months from the date of purchase. Accessories and consumables (tips, tip changer, nozzle cleaner, pump cartridge, coolant hose, coolant filter, O-rings) are excluded from the warranty.

We accept no responsibility for damage caused by incorrect handling or by repairs carried out by third parties not authorised to do so by iM3!

Claims under warranty accompanied by proof of purchase must be sent to the vendor or to an authorised iM3 service partner.

The provision of service under warranty extends neither the warranty period nor any other guarantee period.

24 months warranty

Authorised iM3 service partners

Find your nearest iM3 service partner please contact:

iM3 Pty Ltd - Australia/Asia

21 Chaplin Drive, Lane Cove NSW 2066 Australia

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iM3 Inc. - USA/Canada
iM3 Dental Ltd - Europe